


## Control Syringes Instructions for Use

Please read instructions carefully before use.

### Structure of Control Syringe:

Control Syringe consists of barrel, plunger, handle, piston, male luer connector, etc. (see graph 1)

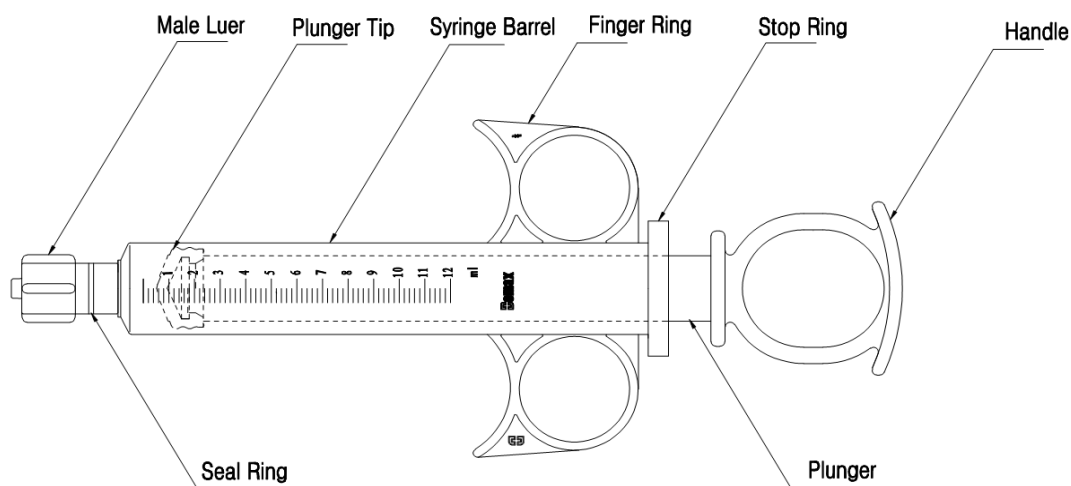

Graph 1

### Indications:

Control syringe is intended to fulfill the market demand, for injection of the contrast media and saline into human body during interventional surgery.

### Instructions for Use:

1. Check that the package and control syringe are not damaged or opened.
2. Push the plunger to the zero graduation.
3. Fill the solution into the control syringe.
4. Push the plunger to eliminate air in the control syringe.
5. Connect the luer connector with a catheter which is inserted into the blood vessel.
6. Push the plunger to transport the solution.

### Warnings:

- Read the instructions before use.
- The device is designed for single use only. Do not de-sterilize and /or reuse.
- Do not use it after expiry date.
- Do not use it if the package is damaged or not well sealed.
- Luer connector must be well connected, incorrect connection or lose connection may cause leaking or air getting into blood vessel.

- Do not inject air into the blood vessel.
- For qualified physicians use only.
- Please dispose the device after use, according to the local legislation.

**Signs and Legends:**



Device contains a sterile fluid path that has been sterilized using ethylene oxide.



Use before date (expiry date)



Do Not Reuse



Caution



Date of manufacture



Batch Code



Qualified physicians use only



Catalogue Number



Contents



Manufacturer



Authorized Representative in the European Community

**Storage:**

Store in a dry, dark, cool place.

The use before date (expiry date) see label.

**Manufacture Information:**

Manufacturer: Beijing Demax Medical Technology Co., Ltd.

Registration Address: First Floor, No.1 Building, No. 8 HongdaBeiLu, BDA, Beijing, China

Manufacturing Address: First Floor, No.1 Building, No. 8 HongdaBeiLu, BDA, Beijing, China

Tel.: 0086-10-67868057

Fax: 0086-10-67860415

Postcode: 100176

Email: [marketing@demaxmedical.com](mailto:marketing@demaxmedical.com)

After-sales Service: Beijing Demax Medical Technology Co., Ltd.

Address: First Floor, No.1 Building, No. 8 HongdaBeiLu, BDA, Beijing, China

Tel.: 0086-10-67868057

Fax: 0086-10-67860415

Postcode: 100176

Email: [marketing@demaxmedical.com](mailto:marketing@demaxmedical.com)

**European Representative:**

Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany

Tel.: 0049-40-2513175

Fax: 0049-40-255726